



# The Dome Director<sup>®</sup>



SELECTION GUIDE

Proudly developed by BrightSky, Australia's leading continence support destination

## Using this Selection Guide

The DomeDirector™ assists healthcare professionals in identifying the type of uridome/sheath that best meets the client's needs when a continence assessment has resulted in the recommendation of uridome/sheath use.

A uridome/sheath is an externally worn device designed to collect urine. It is also referred to as an external catheter. It does not replace the use of an indwelling catheter nor intermittent catheterisation.

Uridomes/sheaths come in varying lengths, widths and adhesive types. One type will not suit everyone. Prior to use, it is important to assess the width, length and skin integrity of the penis correctly.

More information on products listed in this Selection Guide can be found on the BrightSky Australia website. All products can also be ordered through BrightSky Australia.

The development of the BrightSky Australia DomeDirector™ by Kylie Wicks MN (Research), BHLth, Dip App Sc (Nursing), CNC, is acknowledged.

The products listed in this booklet are those that are available through BrightSky Australia at the time of printing. It is subject to change. This booklet will be updated and new products added should these become available from product manufacturers.

---

1. Continence Foundation Australia Retrieved 9 November 2017 from <http://www.continence.org.au/pageskey-statistics.html> 2. Wicks, K., 2015. Investigation of Nursing Knowledge of Catheter Selection following the Introduction of a Catheter Decision Support Tool.

## Contents

How to use The DomeDirector®	4
Helpful Tips	5
Category A	6
Category B	9
Category C	10
Category D	11
Category E	12
Category F	13
Uridome/Sheath Assistive Devices	16

For more information and to order products:  
[www.brightsky.com.au](http://www.brightsky.com.au)  
[orders@brightsky.com.au](mailto:orders@brightsky.com.au)  
1300 88 66 01

# How to use The Dome Director®



## The DomeDirector™ Sizing Guide

### Step 1

Measure the width of the penis using the DomeDirector™ Sizing Guide.

### Step 2

Then measure the length of the flaccid penis with the DomeDirector™ Sizing Guide and note if length variance occurs with the change of position. The uridome should be fitted using the position that provides the most accurate length.

### Step 3

Consider the individual client requirements as listed to the right and then refer to the DomeDirector™ Selection Guide.

The DomeDirector™ has been designed to assist clinicians select a product when a continence management plan has been established. It does not replace a clinician assessment. The DomeDirector™ has been developed on the basis that products perform as indicated by the manufacturer. Application requirements vary and BrightSky recommends that you refer to the manufacturer product information leaflet prior to using a particular uridome/sheath.

Where evidence is not available regarding recommended use, product peer opinion has been taken into account.

Please contact  
BrightSky for  
your brand  
neutral sizing  
tool.



## Helpful Tips

### Adhesion of the uridome/sheath

The length to which a uridome/sheath is to be unrolled prior to its application varies. Always refer to the manufacturer's instructions prior to use.

#### One-Piece

A One-Piece uridome/sheath will either be self-adhesive or require skin glue or adhesion to the penis. Immediately after applying a self-adhesive uridome/sheath, it may be helpful to apply gentle hand pressure for up to 5-10 seconds around it to ensure activation of the adhesive. When using a uridome/sheath that requires glue, only use skin glue designed for this purpose and refer to the manufacturer's instructions for glue application as air time may vary.

#### Two-Piece

A Two-Piece uridome/sheath refers to when uridome/sheath adhesive tape is used to provide the adhesion between the penis and the uridome/sheath. Remember to only use tape specifically designed for uridome/sheath use as applying alternative tapes may result in trauma to the penis. As uridome/sheath tape can be applied in different ways, always read the manufacturer's instructions prior to use.

Note any lotions/moisturisers applied to the penis may interfere with uridome/sheath adhesion. A fenestrated cloth may be of assistance to keep pubic hair out of the way when applying the uridome/sheath.

### Removal of the uridome/sheath

When removing the uridome/sheath, gently roll it from the penis. This may be easier with warm soapy water. It is important to remove all adhesive residue as it may interfere with future successful uridome/sheath adhesion. An adhesive remover may be necessary.

### What type of leg bag should be used with a uridome/sheath?

A uridome/sheath is an external system and a non-sterile bag is connected for drainage collection. The length of bag tubing is dependent on where the person wishes to wear the leg bag, however a certain amount of tubing is required for movement and position change. Always remember to support the leg bag with either straps or a holder to prevent tugging on the uridome/sheath.

As it is an external system, the uridome/sheath can be connected directly to a night bag or a drainage bottle that does not have an anti-reflux valve. In the case of a high pressure void, it may be beneficial not to have an anti-reflux valve as the anti-reflux valve may slow down drainage.

The final consideration when choosing a bag is deciding on the type of tap preferred for ease of emptying.

# Category A

BSA Code	Supplier Code	Description	Self Adhesive	Requires Adhesive	Supplied with Tape	Contains Latex	Shorter than 4cm	Longer than 4cm
143110	5200	Conveen I Piece Latex Self Adhesive Urisheath Standard 25mm	●			●		●
143115	5205	Conveen I Piece Latex Self Adhesive Urisheath Standard 30mm	●			●		●
143120	5210	Conveen I Piece Latex Self Adhesive Urisheath Standard 35mm	●			●		●
143035	5125	Conveen 2 Piece Latex Urisheath Standard 25mm			●	●		●
143040	5130	Conveen 2 Piece Latex Urisheath Standard 30mm			●	●		●
143045	5135	Conveen 2 Piece Latex Urisheath Standard 35mm			●	●		●
144665	22025	Conveen Optima Urisheath Standard Length 25mm	●					●
144670	22030	Conveen Optima Urisheath Standard Length 30mm	●					●
144675	22035	Conveen Optima Urisheath Standard Length 35mm	●					●
144680	22040	Conveen Optima Urisheath Standard Length 40mm	●					●
143005	5021	Conveen Security + 2 Piece Urisheath Standard 21mm			●			●
143010	5025	Conveen Security + 2 Piece Urisheath Standard 25mm			●			●
143015	5030	Conveen Security + 2 Piece Urisheath Standard 30mm			●			●
143020	5035	Conveen Security + 2 Piece Urisheath Standard 35mm			●			●
143025	5040	Conveen Security + 2 Piece Urisheath Standard 40mm			●			●
143155	5221	Conveen Security + Self Adhesive Urisheath Standard 21mm	●					●
143160	5225	Conveen Security + Self Adhesive Urisheath Standard 25mm	●					●
143165	5230	Conveen Security + Self Adhesive Urisheath Standard 30mm	●					●
143170	5235	Conveen Security+ Self Adhesive Urisheath Standard 35mm	●					●
143175	5240	Conveen Security + Self Adhesive Urisheath Standard 40mm	●					●
223065	8000	Freedom Cath Self Adhesive Latex Ext 23mm	●			●		●
223070	8200	Freedom Cath Self Adhesive Latex Ext 28mm	●			●		●
223075	8205	Freedom Cath Self Adhesive Latex Ext 31mm	●			●		●
223080	8400	Freedom Cath Self Adhesive Latex Ext 35mm	●			●		●
223130	25100	Freedom Clear Cath Latex-Free Self Adhesive 23mm	●					●
223135	25200	Freedom Clear Cath Latex-Free Self Adhesive 28mm	●					●
223140	25300	Freedom Clear Cath Latex-Free Self Adhesive 31mm	●					●

# Category A

BSA Code	Supplier Code	Description	Self Adhesive	Requires Adhesive	Supplied with Tape	Contains Latex	Shorter than 4cm	Longer than 4cm
223145	25400	Freedom Clear Cath Latex-Free Self Adhesive 35mm	●					●
223150	25500	Freedom Clear Cath Latex-Free Self Adhesive 40mm	●					●
223485	25190	Freedom Clear LS Silicone Small 23mm	●					●
223490	25290	Freedom Clear LS Silicone Medium 28mm	●					●
223495	25390	Freedom Clear LS Silicone Intermediate 31mm	●					●
223500	25490	Freedom Clear LS Silicone Large 35mm	●					●
223505	25590	Freedom Clear LS Silicone X-Large 40mm	●					●
621070	97525	Inview Standard Sheath - Non Latex - Self Adhesive 25mm	●					●
621075	97529	Inview Standard Sheath - Non Latex - Self Adhesive 29mm	●					●
621080	97532	Inview Standard Sheath - Non Latex - Self Adhesive 32mm	●					●
621085	97536	Inview Standard Sheath - Non Latex - Self Adhesive 36mm	●					●
621090	97541	Inview Standard Sheath - Non Latex - Self Adhesive 41mm	●					●
621025	97625	InView Extra Sheath - Non Latex - Self Adhesive 25mm	●					●
621030	97629	InView Extra Sheath - Non Latex - Self Adhesive 29mm	●					●
621035	97632	Inview Extra Sheath - Non Latex - Self Adhesive 32mm	●					●
621041	97636	Inview Extra Sheath - Non Latex - Self Adhesive 36mm	●					●
621045	97641	Inview Extra Sheath - Non Latex - Self Adhesive 41mm	●					●
223615	97.7501.18	P-Sure Self Adhesive Sheath 18mm	●					●
223620	97.7501.20	P-Sure Self Adhesive Sheath 20mm	●					●
223625	97.7501.22	P-Sure Self Adhesive Sheath 22mm	●					●
223630	97.7501.24	P-Sure Self Adhesive Sheath 24mm	●					●
223635	97.7501.26	P-Sure Self Adhesive Sheath 26mm	●					●
223640	97.7501.28	P-Sure Self Adhesive Sheath 28mm	●					●
223645	97.7501.30	P-Sure Self Adhesive Sheath 30mm	●					●
223650	97.7501.32	P-Sure Self Adhesive Sheath 32mm	●					●
223655	97.7501.35	P-Sure Self Adhesive Sheath 35mm	●					●
223660	97.7501.37	P-Sure Self Adhesive Sheath 37mm	●					●

# Category A

BSA Code	Supplier Code	Description	Self Adhesive	Requires Adhesive	Supplied with Tape	Contains Latex	Shorter than 4cm	Longer than 4cm
223665	97.7501.40	P-Sure Self Adhesive Sheath 40mm	●					●
144055	33301	Rochester Ultra-Flex Urisheath Standard Length 25mm	●					●
144060	33302	Rochester Ultra-Flex Urisheath Standard Length 29mm	●					●
144065	33303	Rochester Ultra-Flex Urisheath Standard Length 32mm	●					●
144070	33304	Rochester Ultra-Flex Urisheath Standard Length 36mm	●					●
144105	36301	Rochester Wide Band Urisheath Self Adhesive Silicone 25mm	●					●
144110	36302	Rochester Wide Band Urisheath Self Adhesive Silicone 29mm	●					●
144115	36303	Rochester Wide Band Urisheath Self Adhesive Silicone 32mm	●					●
144120	36304	Rochester Wide Band Urisheath Self Adhesive Silicone 36mm	●					●
223000	8100	Sayco Active Cath Latex Self Adhesive Ext 23mm	●			●		●
223005	8300	Sayco Active Cath Latex Self Adhesive Ext 28mm	●			●		●
223010	8305	Sayco Active Cath Latex Self Adhesive Ext 31mm	●			●		●
223015	8500	Sayco Active Cath Latex Self Adhesive Ext 35mm	●			●		●
223200	6100	Sayco Clear Advantage with Aloe Self Adhesive Silicone Uridome 23mm	●					●
223205	6200	Sayco Clear Advantage with Aloe Self Adhesive Silicone Uridome 28mm	●					●
223210	6300	Sayco Clear Advantage with Aloe Self Adhesive Silicone Uridome 31mm	●					●
223215	6400	Sayco Clear Advantage with Aloe SelfAdhesive Silicone Uridome 35mm	●					●
461160	U005830000M	Uridrop Latex Uridome 30mm			●	●		●
581005	862	Urisure Latex Uridome 25mm		●		●		●
581010	865	Urisure Latex Uridome 30mm		●		●		●
223370	52013501	Urocare Cath (NoTape) Standard Large 35mm		●		●		●
223425	5221-30	Urocare External Catheter Latex Double Sided Tape 30mm			●	●		●
223430	5221-35	Urocare External Catheter Latex Double Sided Tape 35mm			●	●		●
223435	5221-40	Urocare External Catheter Latex Double Sided Tape 40mm			●	●		●



# Category B

BSA Code	Supplier Code	Description	Self Adhesive	Requires Adhesive	Supplied with Tape	Contains Latex	Shorter than 4cm	Longer than 4cm
143255	22011	Conveen Security + Easifit Urisheath Short Self Adhesive 21mm	●				●	
143260	22012	Conveen Security + Easifit Urisheath Short Self Adhesive 25mm	●				●	
143265	22013	Conveen Security + Easifit Urisheath Short Self Adhesive 30mm	●				●	
143270	22014	Conveen Security + Easifit Urisheath Short Self Adhesive 35mm	●				●	
144685	22121	Conveen Optima Urisheath Short Length 21mm	●				●	
144690	22125	Conveen Optima Urisheath Short Length 25mm	●				●	
144695	22130	Conveen Optima Urisheath Short Length 30mm	●				●	
144700	22135	Conveen Optima Urisheath Short Length 35mm	●				●	
223555	25110	Freedom Clear SS Non Latex Small 23mm	●				●	
223560	25210	Freedom Clear SS Non Latex Medium 28mm	●				●	
223565	25310	Freedom Clear SS Non Latex Intermediate 31mm	●				●	
223570	25410	Freedom Clear SS Non Latex Large 35mm	●				●	
223575	25510	Freedom Clear SS Non Latex X-Large 40mm	●				●	
621000	97425	InView Special Sheath - Non Latex - Self Adhesive 25mm	●				●	
621005	97429	InView Special Sheath - Non Latex - Self Adhesive 29mm	●				●	
621010	97432	InView Special Sheath - Non Latex - Self Adhesive 32mm	●				●	
621015	97436	InView Special Sheath - Non Latex - Self Adhesive 36mm	●				●	
621020	97441	InView Special Sheath - Non Latex - Self Adhesive 41mm	●				●	
144005	32301	Rochester Pop-On Self Adhesive Urisheath Short 25mm	●				●	
144010	32302	Rochester Pop-On Self Adhesive Urisheath Short 29mm	●				●	
144015	32303	Rochester Pop-On Self Adhesive Urisheath Short 32mm	●				●	
144020	32304	Rochester Pop-On Self Adhesive Urisheath Short 36mm	●				●	
144025	32305	Rochester Pop-On Self Adhesive Urisheath Short 41mm	●				●	

# Category C

BSA Code	Supplier Code	Description	Self Adhesive	Requires Adhesive	Supplied with Tape	Contains Latex	Shorter than 4cm	Longer than 4cm
223615	97.7501.18	P-Sure Self Adhesive Sheath 18mm	●					●
223620	97.7501.20	P-Sure Self Adhesive Sheath 20mm	●					●
223625	97.7501.22	P-Sure Self Adhesive Sheath 22mm	●					●
223630	97.7501.24	P-Sure Self Adhesive Sheath 24mm	●					●
223635	97.7501.26	P-Sure Self Adhesive Sheath 26mm	●					●
223640	97.7501.28	P-Sure Self Adhesive Sheath 28mm	●					●
223645	97.7501.30	P-Sure Self Adhesive Sheath 30mm	●					●
223650	97.7501.32	P-Sure Self Adhesive Sheath 32mm	●					●
223655	97.7501.35	P-Sure Self Adhesive Sheath 35mm	●					●
223660	97.7501.37	P-Sure Self Adhesive Sheath 37mm	●					●
223665	97.7501.40	P-Sure Self Adhesive Sheath 40mm	●					●

# Category D

BSA Code	Supplier Code	Description	Self Adhesive	Requires Adhesive	Supplied with Tape	Contains Latex	Shorter than 4cm	Longer than 4cm
223485	25190	Freedom Clear LS Silicone Small 23mm	●					●
223490	25290	Freedom Clear LS Silicone Medium 28mm	●					●
223495	25390	Freedom Clear LS Silicone Intermediate 31mm	●					●
223500	25490	Freedom Clear LS Silicone Large 35mm	●					●
223505	25590	Freedom Clear LS Silicone X-Large 40mm	●					●
621025	97625	InView Extra Sheath - Non Latex - Self Adhesive 25mm	●					●
621030	97629	InView Extra Sheath - Non Latex - Self Adhesive 29mm	●					●
621035	97632	Inview Extra Sheath - Non Latex - Self Adhesive 32mm	●					●
621041	97636	Inview Extra Sheath - Non Latex - Self Adhesive 36mm	●					●
621045	97641	Inview Extra Sheath - Non Latex - Self Adhesive 41mm	●					●
144105	36301	Rochester Wide Band Urisheath Self Adhesive Silicone 25mm	●					●
144110	36302	Rochester Wide Band Urisheath Self Adhesive Silicone 29mm	●					●
144115	36303	Rochester Wide Band Urisheath Self Adhesive Silicone 32mm	●					●
144120	36304	Rochester Wide Band Urisheath Self Adhesive Silicone 36mm	●					●
223000	8100	Sayco Active Cath Latex Self Adhesive Ext 23mm	●			●		●
223005	8300	Sayco Active Cath Latex Self Adhesive Ext 28mm	●			●		
223010	8305	Sayco Active Cath Latex Self Adhesive Ext 31mm	●			●		
223015	8500	Sayco Active Cath Latex Self Adhesive Ext 35mm	●			●		

# Category E

BSA Code	Supplier Code	Description	Self Adhesive	Requires Adhesive	Supplied with Tape	Contains Latex	Shorter than 4cm	Longer than 4cm
144665	22025	Conveen Optima Urisheath Standard Length 25mm	●					●
144670	22030	Conveen Optima Urisheath Standard Length 30mm	●					●
144675	22035	Conveen Optima Urisheath Standard Length 35mm	●					●
144680	22040	Conveen Optima Urisheath Standard Length 40mm	●					●
144685	22121	Conveen Optima Urisheath Short Length 21mm	●				●	
144690	22125	Conveen Optima Urisheath Short Length 25mm	●				●	
144695	22130	Conveen Optima Urisheath Short Length 30mm	●				●	
144700	22135	Conveen Optima Urisheath Short Length 35mm	●				●	

# Category F

BSA Code	Supplier Code	Description	Self Adhesive	Requires Adhesive	Supplied with Tape	Contains Latex	Shorter than 4cm	Longer than 4cm
223130	25100	Freedom Clear Latex-Free Self Adhesive 23mm	●					●
223135	25200	Freedom Clear Latex-Free Self Adhesive 28mm	●					●
223140	25300	Freedom Clear Cath Latex-Free Self Adhesive 31mm	●					●
223145	25400	Freedom Clear Cath Latex-Free Self Adhesive 35mm	●					●
223150	25500	Freedom Clear Cath Latex-Free Self Adhesive 40mm	●					●
223485	25190	Freedom Clear LS Silicone Small 23mm	●					●
223490	25290	Freedom Clear LS Silicone Medium 28mm	●					●
223495	25390	Freedom Clear LS Silicone Intermediate 31mm	●					●
223500	25490	Freedom Clear LS Silicone Large 35mm	●					●
223505	25590	Freedom Clear LS Silicone X-Large 40mm	●					●
223555	25110	Freedom Clear SS Non Latex Small 23mm	●				●	
223560	25210	Freedom Clear SS Non Latex Medium 28mm	●				●	
223570	25410	Freedom Clear SS Non Latex Large 35mm	●				●	
223575	25510	Freedom Clear SS Non Latex X-Large 40mm	●				●	
621070	97525	Inview Standard Sheath - Non Latex - Self Adhesive 25mm	●					●
621075	97529	Inview Standard Sheath - Non Latex - Self Adhesive 29mm	●					●
621080	97532	Inview Standard Sheath - Non Latex - Self Adhesive 32mm	●					●
621085	97536	Inview Standard Sheath - Non Latex - Self Adhesive 36mm	●					●
621090	97541	Inview Standard Sheath - Non Latex - Self Adhesive 41mm	●					●
621025	97625	InView Extra Sheath - Non Latex - Self Adhesive 25mm	●					●
621030	97629	InView Extra Sheath - Non Latex - Self Adhesive 29mm	●					●
621035	97632	Inview Extra Sheath - Non Latex - Self Adhesive 32mm	●					●
621041	97636	Inview Extra Sheath - Non Latex - Self Adhesive 36mm	●					●
621045	97641	Inview Extra Sheath - Non Latex - Self Adhesive 41mm	●					●
621000	97425	InView Special Sheath - Non Latex - Self Adhesive 25mm	●				●	
621005	97429	InView Special Sheath - Non Latex - Self Adhesive 29mm	●				●	
621010	97432	InView Special Sheath - Non Latex - Self Adhesive 32mm	●				●	

# Category F

BSA Code	Supplier Code	Description	Self Adhesive	Requires Adhesive	Supplied with Tape	Contains Latex	Shorter than 4cm	Longer than 4cm
621015	97436	InView Special Sheath - Non Latex - Self Adhesive 36mm	●				●	
621020	97441	InView Special Sheath - Non Latex - Self Adhesive 41mm	●				●	
144005	32301	Rochester Pop-On Self Adhesive Urisheath Short 25mm	●				●	
144010	32302	Rochester Pop-On Self Adhesive Urisheath Short 29mm	●				●	
144015	32303	Rochester Pop-On Self Adhesive Urisheath Short 32mm	●				●	
144020	32304	Rochester Pop-On Self Adhesive Urisheath Short 36mm	●				●	
144025	32305	Rochester Pop-On Self Adhesive Urisheath Short 41mm	●				●	
144055	33301	Rochester Ultra-Flex Urisheath Standard Length 25mm	●					●
144060	33302	Rochester Ultra-Flex Urisheath Standard Length 29mm	●					●
144065	33303	Rochester Ultra-Flex Urisheath Standard Length 32mm	●					●
144070	33304	Rochester Ultra-Flex Urisheath Standard Length 36mm	●					●
144105	36301	Rochester Wide Band Urisheath Self Adhesive Silicone 25mm	●					●
144110	36302	Rochester Wide Band Urisheath Self Adhesive Silicone 29mm	●					●
144115	36303	Rochester Wide Band Urisheath Self Adhesive Silicone 32mm	●					●
144120	36304	Rochester Wide Band Urisheath Self Adhesive Silicone 36mm	●					●
223200	6100	Sayco Clear Advantage with Aloe Self Adhesive Silicone Uridome 23mm	●					●
223205	6200	Sayco Clear Advantage with Aloe Self Adhesive Silicone Uridome 28mm	●					●
223210	6300	Sayco Clear Advantage with Aloe Self Adhesive Silicone Uridome 31mm	●					●
223215	6400	Sayco Clear Advantage with Aloe Self Adhesive Silicone Uridome 35mm	●					●
144015	32303	Rochester Pop-On Self Adhesive Urisheath Short 32mm	●				●	
144020	32304	Rochester Pop-On Self Adhesive Urisheath Short 36mm	●				●	
144025	32305	Rochester Pop-On Self Adhesive Urisheath Short 41mm	●				●	
144055	33301	Rochester Ultra-Flex Urisheath Standard Length 25mm	●					●
144060	33302	Rochester Ultra-Flex Urisheath Standard Length 29mm	●					●



BSA Code	Supplier Code	Description	Self Adhesive	Requires Adhesive	Supplied with Tape	Contains Latex	Shorter than 4cm	Longer than 4cm
144065	33303	Rochester Ultra-Flex Urisheath Standard Length 32mm	●					●
144070	33304	Rochester Ultra-Flex Urisheath Standard Length 36mm	●					●
144105	36301	Rochester Wide Band Urisheath Self Adhesive Silicone 25mm	●					●
144110	36302	Rochester Wide Band Urisheath Self Adhesive Silicone 29mm	●					●
144115	36303	Rochester Wide Band Urisheath Self Adhesive Silicone 32mm	●					●
144120	36304	Rochester Wide Band Urisheath Self Adhesive Silicone 36mm	●					●
223200	6100	Sayco Clear Advantage with Aloe Self Adhesive Silicone Uridome 23mm	●					●
223205	6200	Sayco Clear Advantage with Aloe Self Adhesive Silicone Uridome 28mm	●					●
223210	6300	Sayco Clear Advantage with Aloe Self Adhesive Silicone Uridome 31mm	●					●
223215	6400	Sayco Clear Advantage with Aloe Self Adhesive Silicone Uridome 35mm	●					●

# Uridome/Sheath Assistive Devices

BSA Code	Supplier Code	Description
621095	9873	Hollister Retracted Penis Pouch with Drainage
223600	50.2	Sauer Glue Standard Adhesive 2% Resin 28g Tube
223605	50.22	Sauer Glue Strong Adhesive 12% Resin 28g Tube
225820	5600 x 1	Sayco Adhesive Remover Pads 50s Box
228005	5004-03	Uro-Bond IV Brush on Non Flammable Adhesive 88.7ml
223280	500330	Urocare Latex Sheath 30mm
223270	500335	Urocare Latex Sheath 35mm
227055	4399	Urocare Male Urinal Kit Standard Small (contains: Suspensory Garment, Sheath, Bottom Drain Valve, Adaptors & Thumb Clamp)
225115	6016	Urocare Non Sterile Rubber Extension Tube with Connector 45cm length 8mmID
225166	6061	Urocare Non Sterile Clear Vinyl Extension Tube 1.5m length 7mmI.D
225920	6008	Urocare White Rubber Drainage Tubing 3m 8mm I.D
225160	SB6060s	Strobus Sterile Extension Tubing 1.5m Ea
225415	6004	Urocare Replacement Standard Bottom Drain Valve
227000	4409	Urocare Replacement Urinal Sheath 20-40mmID 10cm Long
227005	4410	Urocare Replacement Urinal Sheath 20-40mmID 18cm Long
225000	5100	Urocare Single Sided Adhesive Foam Strips
225105	6009	Urocare Tubing Connector Small 8mm O.D 6cm
225920	6008	Urocare White Rubber Tubing 30cm
225770	5200	Urofoam 2 Double Sided Adhesive Strip

At BrightSky, we're about supporting you on your unique path through wellness, to help you maintain and improve your health and support you to live your best life.

With commitment to delivering quality products, supported by our dedicated customer service team, we are continuously striving to go the extra mile for our customers. We aim to support our community with genuine wellness solutions that are focused on improving quality of life.

We aspire to play a vital role in elevating the lives of our customers; the people we support, live and work with; and the wider community through the extensive range of products we deliver and the crucial services the charitable organisation we're part of, provides.

*live well*

---

**BrightSky Australia is an NDIS registered provider.**

# Discover how BrightSky can support you.

[www.brightskyclinical.com.au](http://www.brightskyclinical.com.au)  
[www.brightsky.com.au](http://www.brightsky.com.au)  
[orders@brightsky.com.au](mailto:orders@brightsky.com.au)  
1300 88 66 01

**Offices**  
6 Holker Street  
Newington NSW 2127  
32 Benison Road  
Winnellie NT 0820  
Unit7, 576 Boundary Road  
Archerfield QLD 4108

**Special Assistance**  
Interpreter Service 131 450  
National Relay Service  
(for the deaf, hearing or  
speech impaired)  
**Fax**  
1300 88 66 02

**TTY/Voice**  
133 677  
**SSR (Speak and Listen)**  
1300 555 727  
**Post**  
PO Box 6347  
Silverwater DC NSW 1811

## Need advice?

Our practical, portable tools such as PadNavigator® 2.0, The Catheter Compass 2.0™, The DomeDirector®, The Drainage Companion™ and Traversing Transanal Irrigation can help you find and recommend the right product for your customers.

