

Dome Director®

PRODUCT GUIDE



Proudly developed by BrightSky, Australia's leading continence support destination

BrightSky[®]
AUSTRALIA

Using this Product Guide

The DomeDirector® assists healthcare professionals in identifying the type of uridome/sheath that best meets the client's needs when a continence assessment has resulted in the recommendation of uridome/sheath use.

A uridome/sheath is an externally worn device designed to collect urine. It is also referred to as an external catheter. It does not replace the use of an indwelling catheter nor intermittent catheterisation.

Uridomes/sheaths come in varying lengths, widths and adhesive types. One type will not suit everyone. Prior to use, it is important to assess the width, length and skin integrity of the penis correctly.

More information on products listed in this Product Guide can be found on the BrightSky Australia website. All products can also be ordered through BrightSky Australia.

The products listed in this Guide are those that were available through BrightSky Australia at the time of printing and are subject to change. This Product Guide will be updated and new products added as they become available from product manufacturers.

The development of the BrightSky Australia DomeDirector® by Kylie Wicks MN (Research), BHLth, Dip App Sc (Nursing), CNC, is acknowledged.

Contents

| | |
|----------------------------------|----|
| Using The DomeDirector® | 4 |
| Category 1 | 6 |
| Category 2 | 9 |
| Category 3 | 10 |
| Category 4 | 10 |
| Category 5 | 11 |
| Category 6 | 11 |
| Uridome/Sheath Assistive Devices | 13 |

How to use the Dome Director®

Step 1

Measure the width of the penis using the DomeDirector® Sizing Guide.

Step 2

Then measure the length of the flaccid penis with the DomeDirector® Sizing Guide and note if length variance occurs with the change of position. The uridome should be fitted using the position that provides the most accurate length.

Step 3

Consider the individual client requirements as listed to the right and then refer to the DomeDirector® Product Guide.

Category 1

Penis length longer than 4cm.
No further individual client requirements.

Category 2

Penis length shorter than 4cm.
No further individual client requirements.

Category 3

Urine Collection from a high pressure void.

Category 4

Active client requiring wider adhesive band (for sports or similar use).

Category 5

Dexterity issues require pull tab applicator to assist application.

Category 6

Penis skin requires monitoring.



The DomeDirector® has been designed to assist clinicians select a product when a continence management plan has been established. It does not replace a clinician assessment. The DomeDirector® has been developed on the basis that products perform as indicated by the manufacturer. Application requirements vary and BrightSky recommends that you refer to the manufacturer product information leaflet prior to using a particular uridome/sheath.

Where evidence is not available regarding recommended use, product peer opinion has been taken into account.

Please contact
BrightSky
for your
brand neutral
sizing tool.

Helpful Tips

Adhesion of the uridome/sheath

The length to which a uridome/sheath is to be unrolled prior to its application varies. Always refer to the manufacturer's instructions prior to use.

One-Piece

A One-Piece uridome/sheath will either be self-adhesive or require skin glue or adhesion to the penis. Immediately after applying a self-adhesive uridome/sheath, it may be helpful to apply gentle hand pressure for up to 5-10 seconds around it to ensure activation of the adhesive. When using a uridome/sheath that requires glue, only use skin glue designed for this purpose and refer to the manufacturer's instructions for glue application as air time may vary.

Two-Piece

A Two-Piece uridome/sheath refers to when uridome/sheath adhesive tape is used to provide the adhesion between the penis and the uridome/sheath. Remember to only use tape specifically designed for uridome/sheath use as applying alternative tapes may result in trauma to the penis. As uridome/sheath tape can be applied in different ways, always read the manufacturer's instructions prior to use.

Note any lotions/moisturisers applied to the penis may interfere with uridome/sheath adhesion. A fenestrated cloth may be of assistance to keep pubic hair out of the way when applying the uridome/sheath.

Removal of the uridome/sheath

When removing the uridome/sheath, gently roll it from the penis. This may be easier with warm soapy water. It is important to remove all adhesive residue as it may interfere with future successful uridome/sheath adhesion. An adhesive remover may be necessary.

What type of leg bag should be used with a uridome/sheath?

A uridome/sheath is an external system and a non-sterile bag is connected for drainage collection. The length of bag tubing is dependent on where the person wishes to wear the leg bag, however a certain amount of tubing is required for movement and position change. Always remember to support the leg bag with either straps or a holder to prevent tugging on the uridome/sheath.

As it is an external system, the uridome/sheath can be connected directly to a night bag or a drainage bottle that does not have an anti-reflux valve. In the case of a high pressure void, it may be beneficial not to have an anti-reflux valve as the anti-reflux valve may slow down drainage.

The final consideration when choosing a bag is deciding on the type of tap preferred for ease of emptying.

Uridomes/Sheaths



| Category 1 | | | | | | | | | |
|---------------------|-------------------|---|---------------|-------------------|--------------------|----------------|------------------|-----------------|---|
| BrightSky Item Code | Manufacturer Code | Description | Self Adhesive | Requires Adhesive | Supplied with Tape | Contains Latex | Shorter than 4cm | Longer than 4cm | |
| 143005 | 5021 | Conveen Security + 2 Piece Urisheath Standard 21mm | | | ● | | | | ● |
| 143010 | 5025 | Conveen Security + 2 Piece Urisheath Standard 25mm | | | ● | | | | ● |
| 143015 | 5030 | Conveen Security + 2 Piece Urisheath Standard 30mm | | | ● | | | | ● |
| 143020 | 5035 | Conveen Security + 2 Piece Urisheath Standard 35mm | | | ● | | | | ● |
| 143025 | 5040 | Conveen Security + 2 Piece Urisheath Standard 40mm | | | ● | | | | ● |
| 143035 | 5125 | Conveen 2 Piece Urisheath Standard 25mm | | | ● | ● | | | ● |
| 143040 | 5130 | Conveen 2 Piece Urisheath Standard 30mm | | | ● | ● | | | ● |
| 143045 | 5135 | Conveen 2 Piece Urisheath Standard 35mm | | | ● | ● | | | ● |
| 143110 | 5200 | Conveen 1 Piece Urisheath Standard 25mm | ● | | | ● | | | ● |
| 143115 | 5205 | Conveen 1 Piece Urisheath Standard 30mm | ● | | | ● | | | ● |
| 143120 | 5210 | Conveen 1 Piece Urisheath Standard 35mm | ● | | | ● | | | ● |
| 143155 | 5221 | Conveen Security + Urisheath Standard 21mm | ● | | | | | | ● |
| 143160 | 5225 | Conveen Security + Urisheath Standard 25mm | ● | | | | | | ● |
| 143165 | 5230 | Conveen Security + Urisheath Standard 30mm | ● | | | | | | ● |
| 143170 | 5235 | Conveen Security + Urisheath Standard 35mm | ● | | | | | | ● |
| 143175 | 5240 | Conveen Security + Urisheath Standard 40mm | ● | | | | | | ● |
| 144055 | 33301 | Rochester Ultraflex Silicon Urisheath Standard 25mm | ● | | | | | | ● |
| 144060 | 33302 | Rochester Ultraflex Silicon Urisheath Standard 29mm | ● | | | | | | ● |
| 144065 | 33303 | Rochester Ultraflex Silicon Urisheath Standard 32mm | ● | | | | | | ● |
| 144070 | 33304 | Rochester Ultraflex Silicon Urisheath Standard 36mm | ● | | | | | | ● |
| 144075 | 33305 | Rochester Ultraflex Silicon Urisheath Standard 41mm | ● | | | | | | ● |
| 144105 | 36301 | Rochester Wide Band Silicon Urisheath 25mm | ● | | | | | | ● |
| 144110 | 36302 | Rochester Wide Band Silicon Urisheath 29mm | ● | | | | | | ● |
| 144120 | 36304 | Rochester Wide Band Silicon Urisheath 36mm | ● | | | | | | ● |
| 144665 | 22025 | Conveen Optima Urisheath Standard Length 25mm | ● | | | | | | ● |
| 144667 | 22028 | Conveen Optima Urisheath Standard length 28mm | ● | | | | | | ● |

Category 1

| BrightSky Item Code | Manufacturer Code | Description | Self Adhesive | Requires Adhesive | Supplied with Tape | Contains Latex | Shorter than 4cm | Longer than 4cm |
|---------------------|-------------------|--|---------------|-------------------|--------------------|----------------|------------------|-----------------|
| 144670 | 22030 | Conveen Optima Urisheath Standard Length 30mm | ● | | | | | ● |
| 144675 | 22035 | Conveen Optima Urisheath Standard Length 35mm | ● | | | | | ● |
| 144680 | 22040 | Conveen Optima Urisheath Standard Length 40mm | ● | | | | | ● |
| 223000 | 8100 | Sayco Active Catheter External 23mm | ● | | | ● | ● | |
| 223005 | 8300 | Sayco Active Catheter External 28mm | ● | | | ● | ● | |
| 223010 | 8305 | Sayco Active Catheter External 31mm | ● | | | ● | ● | |
| 223015 | 8500 | Sayco Active Catheter External 35mm | ● | | | ● | ● | |
| 223065 | 8000 | Freedom Catheter External 23mm | ● | | | ● | | ● |
| 223070 | 8200 | Freedom Catheter External 28mm | ● | | | ● | | ● |
| 223075 | 8205 | Freedom Catheter External 31mm | ● | | | ● | | ● |
| 223080 | 8400 | Freedom Catheter External 35mm | ● | | | ● | | ● |
| 223141 | SB25300 | Strobus Silicone External Catheter Intermediate 31mm | ● | | | | | ● |
| 223270 | 500335 | Urocare Urocon External Catheter 35mm | | ● | | ● | | ● |
| 223366 | 52013050 | Uro-Cath 30mm | | ● | | ● | | ● |
| 223371 | 52013550 | Uro Cath 35mm | | ● | | ● | | ● |
| 223430 | 52213550/5200 | Uro-Cath External Catheter 35mm | | | ● | ● | | ● |
| 223435 | 52214050/5200 | Uro-Cath External Catheter 40mm | | | ● | ● | | ● |
| 223486 | SB25190 | Strobus Silicone External Catheter Long Sheath Small 23mm | ● | | | | | ● |
| 223501 | SB25490 | Strobus Silicone External Catheter Long Sheath Large 35mm | ● | | | | | ● |
| 223556 | SB25110 | Strobus Silicone External Catheter Short Sheath Small 23mm | ● | | | | | ● |
| 223571 | SB25410 | Strobus Silicone External Catheter Short Sheath Large 35mm | ● | | | | | ● |
| 223635 | 97.7501.26 | P-Sure Strong Sheath 26mm Blue | ● | | | | | ● |
| 223645 | 97.7501.30 | P-Sure Strong Sheath 30mm Blue | ● | | | | | ● |
| 223650 | 97.7501.32 | P-Sure Strong Sheath 32mm Blue | ● | | | | | ● |
| 223655 | 97.7501.35 | P-Sure Strong Sheath 35mm Blue | ● | | | | | ● |
| 229130 | 53.3 | Latex Sheath Non-Adhesive 30mm | | ● | | ● | | ● |

Uridomes/Sheaths



| Category 1 | | | | | | | Self Adhesive | Requires Adhesive | Supplied with Tape | Contains Latex | Shorter than 4cm | Longer than 4cm |
|------------|-------|---|--|--|--|--|---------------|-------------------|--------------------|----------------|------------------|-----------------|
| 229145 | 53.35 | Latex Sheath Non-Adhesive 35mm | | | | | | ● | | ● | | ● |
| 621025 | 97625 | Inview Extra Sheath 25mm | | | | | ● | | | | | ● |
| 621030 | 97629 | Inview Extra Sheath 29mm | | | | | ● | | | | | ● |
| 621035 | 97632 | Inview Extra Sheath 32mm | | | | | ● | | | | | ● |
| 621041 | 97636 | Inview Extra Sheath 36mm | | | | | ● | | | | | ● |
| 621045 | 97641 | Inview Extra Sheath 41mm | | | | | ● | | | | | ● |
| 621070 | 97525 | Inview Sheath Standard 25mm | | | | | ● | | | | | ● |
| 621075 | 97529 | Inview Sheath Standard 29mm | | | | | ● | | | | | ● |
| 621080 | 97532 | Inview Sheath Standard 32mm | | | | | ● | | | | | ● |
| 621085 | 97536 | Inview Sheath Standard 36mm | | | | | ● | | | | | ● |
| 621090 | 97541 | Inview Sheath Standard 41mm | | | | | ● | | | | | ● |
| 999097 | 21026 | Mens Liberty Acute External Male Catheter | | | | | ● | | | | | ● |

Category 2

| BrightSky Item Code | Manufacturer Code | Description | Self Adhesive | Requires Adhesive | Supplied with Tape | Contains Latex | Shorter than 4cm | Longer than 4cm |
|---------------------|-------------------|---|---------------|-------------------|--------------------|----------------|------------------|-----------------|
| 143255 | 22011 | Conveen Security + Easifit Urisheath 21mm | ● | | | | | ● |
| 143260 | 22012 | Conveen Security + Easifit Urisheath 25mm | ● | | | | | ● |
| 143265 | 22013 | Conveen Security + Easifit Urisheath 30mm | ● | | | | | ● |
| 143270 | 22014 | Conveen Security + Easifit Urisheath 35mm | ● | | | | | ● |
| 144005 | 32301 | Rochester Pop-On Urisheath 25mm | ● | | | | | ● |
| 144010 | 32302 | Rochester Pop-On Urisheath 29mm | ● | | | | | ● |
| 144015 | 32303 | Rochester Pop-On Urisheath 32mm | ● | | | | | ● |
| 144020 | 32304 | Rochester Pop-On Urisheath 36mm | ● | | | | | ● |
| 144685 | 22121 | Conveen Optima Urisheath 21mm | ● | | | | | ● |
| 144690 | 22125 | Conveen Optima Urisheath 25mm | ● | | | | | ● |
| 144695 | 22130 | Conveen Optima Urisheath 30mm | ● | | | | | ● |
| 144700 | 22135 | Conveen Optima Urisheath 35mm | ● | | | | | ● |
| 621000 | 97425 | Inview Special Sheath 25mm | ● | | | | | ● |
| 621005 | 97429 | Inview Special Sheath 29mm | ● | | | | | ● |
| 621010 | 97432 | Inview Special Sheath 32mm | ● | | | | | ● |
| 621015 | 97436 | Inview Special Sheath 36mm | ● | | | | | ● |
| 621020 | 97441 | Inview Special Sheath 41mm | ● | | | | | ● |

Uridomes/Sheaths



Category 3

| BrightSky Item Code | Manufacturer Code | Description | Self Adhesive | Requires Adhesive | Supplied with Tape | Contains Latex | Shorter than 4cm | Longer than 4cm |
|---------------------|-------------------|--------------------------------|---------------|-------------------|--------------------|----------------|------------------|-----------------|
| 223635 | 97.7501.26 | P-Sure Strong Sheath 26mm Blue | ● | | | | | ● |
| 223645 | 97.7501.30 | P-Sure Strong Sheath 30mm Blue | ● | | | | | ● |
| 223650 | 97.7501.32 | P-Sure Strong Sheath 32mm Blue | ● | | | | | ● |
| 223655 | 97.7501.35 | P-Sure Strong Sheath 35mm Blue | ● | | | | | ● |

Category 4

| BrightSky Item Code | Manufacturer Code | Description | Self Adhesive | Requires Adhesive | Supplied with Tape | Contains Latex | Shorter than 4cm | Longer than 4cm |
|---------------------|-------------------|--|---------------|-------------------|--------------------|----------------|------------------|-----------------|
| 144076 | 36305 | Bard Male External Catheter Wideband Extra Large | ● | | | | | ● |
| 144115 | 36303 | Rochester Wide Band Silicon Urisheath 32mm | ● | | | | | ● |
| 223000 | 8100 | Sayco Active Catheter External 23mm | ● | | | ● | ● | |
| 223005 | 8300 | Sayco Active Catheter External 28mm | ● | | | ● | ● | |
| 223010 | 8305 | Sayco Active Catheter External 31mm | ● | | | ● | ● | |
| 223015 | 8500 | Sayco Active Catheter External 35mm | ● | | | ● | ● | |
| 621025 | 97625 | Inview Extra Sheath 25mm | ● | | | | | ● |
| 621030 | 97629 | Inview Extra Sheath 29mm | ● | | | | | ● |
| 621035 | 97632 | Inview Extra Sheath 32mm | ● | | | | | ● |
| 621041 | 97636 | Inview Extra Sheath 36mm | ● | | | | | ● |
| 621045 | 97641 | Inview Extra Sheath 41mm | ● | | | | | ● |

Category 5

| BrightSky Item Code | Manufacturer Code | Description | Self Adhesive | Requires Adhesive | Supplied with Tape | Contains Latex | Shorter than 4cm | Longer than 4cm |
|---------------------|-------------------|---|---------------|-------------------|--------------------|----------------|------------------|-----------------|
| 144665 | 22025 | Conveen Optima Urisheath Standard Length 25mm | ● | | | | | ● |
| 144667 | 22028 | Conveen Optima Urisheath Standard length 28mm | ● | | | | | ● |
| 144670 | 22030 | Conveen Optima Urisheath Standard Length 30mm | ● | | | | | ● |
| 144675 | 22035 | Conveen Optima Urisheath Standard Length 35mm | ● | | | | | ● |
| 144680 | 22040 | Conveen Optima Urisheath Standard Length 40mm | ● | | | | | ● |
| 144685 | 22121 | Conveen Optima Urisheath 21mm | ● | | | | ● | |
| 144690 | 22125 | Conveen Optima Urisheath 25mm | ● | | | | ● | |
| 144695 | 22130 | Conveen Optima Urisheath 30mm | ● | | | | ● | |
| 144700 | 22135 | Conveen Optima Urisheath 35mm | ● | | | | ● | |

Category 6

| BrightSky Item Code | Manufacturer Code | Description | Self Adhesive | Requires Adhesive | Supplied with Tape | Contains Latex | Shorter than 4cm | Longer than 4cm |
|---------------------|-------------------|---|---------------|-------------------|--------------------|----------------|------------------|-----------------|
| 144005 | 32301 | Rochester Pop-On Urisheath 25mm | ● | | | | ● | |
| 144010 | 32302 | Rochester Pop-On Urisheath 29mm | ● | | | | ● | |
| 144015 | 32303 | Rochester Pop-On Urisheath 32mm | ● | | | | ● | |
| 144020 | 32304 | Rochester Pop-On Urisheath 36mm | ● | | | | ● | |
| 144055 | 33301 | Rochester Ultraflex Silicon Urisheath Standard 25mm | ● | | | | ● | |
| 144060 | 33302 | Rochester Ultraflex Silicon Urisheath Standard 29mm | ● | | | | | ● |
| 144065 | 33303 | Rochester Ultraflex Silicon Urisheath Standard 32mm | ● | | | | | ● |
| 144070 | 33304 | Rochester Ultraflex Silicon Urisheath Standard 36mm | ● | | | | | ● |
| 144075 | 33305 | Rochester Ultraflex Silicon Urisheath Standad 41mm | ● | | | | | ● |
| 144105 | 36301 | Rochester Wide Band Silicon Urisheath 25mm | ● | | | | | ● |
| 144110 | 36302 | Rochester Wide Band Silicon Urisheath 29mm | ● | | | | | ● |
| 144120 | 36304 | Rochester Wide Band Silicon Urisheath 36mm | ● | | | | | ● |

Uridomes/Sheaths



Category 6

| BrightSky Item Code | Manufacturer Code | Description | Self Adhesive | Requires Adhesive | Supplied with Tape | Contains Latex | Shorter than 4cm | Longer than 4cm |
|---------------------|-------------------|--|---------------|-------------------|--------------------|----------------|------------------|-----------------|
| 229375 | SB25210 | Strobus Silicone External Catheter Medium 28mm | ● | | | | ● | |
| 229380 | SB25290 | Strobus Silicone External Catheter Medium 28mm | ● | | | | | ● |
| 229385 | SB25390 | Strobus Silicone External Catheter Intermediate 31mm | ● | | | | | ● |
| 621025 | 97625 | Inview Extra Sheath 25mm | ● | | | | | ● |
| 621030 | 97629 | Inview Extra Sheath 29mm | ● | | | | | ● |
| 621035 | 97632 | Inview Extra Sheath 32mm | ● | | | | | ● |
| 621041 | 97636 | Inview Extra Sheath 36mm | ● | | | | | ● |
| 621045 | 97641 | Inview Extra Sheath 41mm | ● | | | | | ● |
| 621070 | 97525 | Inview Sheath Standard 25mm | ● | | | | | ● |
| 621075 | 97529 | Inview Sheath Standard 29mm | ● | | | | | ● |
| 621080 | 97532 | Inview Sheath Standard 32mm | ● | | | | | ● |
| 621085 | 97536 | Inview Sheath Standard 36mm | ● | | | | | ● |
| 621090 | 97541 | Inview Sheath Standard 41mm | ● | | | | | ● |
| 974305 | 3101 | Ugo Sheath Standard Silicone 28mm | ● | | | | | ● |
| 974310 | 3102 | Ugo Sheath Standard Silicone 30mm | ● | | | | | ● |
| 974345 | 3109 | Ugo Sheath Pop-on 32mm | ● | | | | ● | |
| 994087 | GCU201412 | Paralogic Male External Catheter 28mm | ● | | | | | ● |
| 994088 | GCU201413 | Paralogic Male External Catheter 31mm | ● | | | | | ● |

Uridome/Sheath Assistive Devices

| BrightSky Item Code | Manufacturer Code | Description |
|---------------------|-------------------|---|
| 223600 | 50.2 | Sauer Glue Standard Adhesive 2% Resin 28g |
| 223605 | 50.22 | Sauer Glue Adhesive 12% Resin Strong 28g |
| 225000 | 5100 | Urocare Single Sided Adhesive Foam Strips |
| 225105 | 6009 | Urocare Tubing Connector Small 8mm Outer Diameter 6cm |
| 225115 | 6016 | Urocare Non Sterile Rubber Extension Tube With Connector 45cm 8mm |
| 225160 | SB6060s | Strobus Sterile Extension Tubing 1.5m |
| 225166 | 6061 | Urocare Non-Sterile Vinyl Extension Drain Bottle Tube 1.5m 7mm |
| 225415 | 6004 | Urocare Replacement Standard Bottom Drain Valve |
| 225770 | 5200 | Urofoam 2 Double Sided Adhesive Strip |
| 225820 | 5600 | Sayco Adhesive Remover Pads |
| 228006 | 500003 | Uro-Bond 3 Silicone Brush On Silicone Adhesive 88.7ml |
| 228011 | 5000015 | Uro-Bond 3 Silicone Brush On Silicone Adhesive 44.4ml |

At BrightSky, we're about supporting you on your unique path through wellness, to help you maintain and improve your health and support you to live your best life.

With commitment to delivering quality products, supported by our dedicated customer service team, we are continuously striving to go the extra mile for our customers. We aim to support our community with genuine wellness solutions that are focused on improving quality of life.

We aspire to play a vital role in elevating the lives of our customers; the people we support, live and work with; and the wider community through the extensive range of products we deliver and the crucial services the charitable organisation we're part of, provides.

live well

**BrightSky Australia is an NDIS
registered provider.**

Discover how BrightSky can support you.

For more information and to order...



BrightSky
www.brightsky.com.au
orders@brightsky.com.au
1300 88 66 01

For more information and to order for NDIS participants...



BrightSky NDIS
www.ndis.brightsky.com.au
ndis@brightsky.com.au
1300 88 66 01

For Clinical Tools including practical guides and advice...



BrightSky Clinical
View our Clinical Tools to help problem-solve the specific needs of patients quickly and safely.
www.brightskyclinical.com.au

BrightSky
AUSTRALIA the social enterprise of Forward Ability Support

BrightSky Australia is an
NDIS registered provider

